

Home Care Technical Specification

1 1/2" L-Shaped Concealed Screw Grab Bars – Peened Stainless Steel

8990

Stock Numbers:

8992 – 16" x 32" L-Shape Grab Bar

Stainless Steel Peened Finish
Concealed Screw Mounting
1 1/2" Diameter


8994 – 24" x 36" L-Shape Grab Bar

Stainless Steel Peened Finish
Concealed Screw Mounting
1 1/2" Diameter Bar

8996 – 24" x 48" L-Shape Grab Bar

Stainless Steel Peened Finish
Concealed Screw mounting
1 1/2" Diameter Bar

Dimensions:

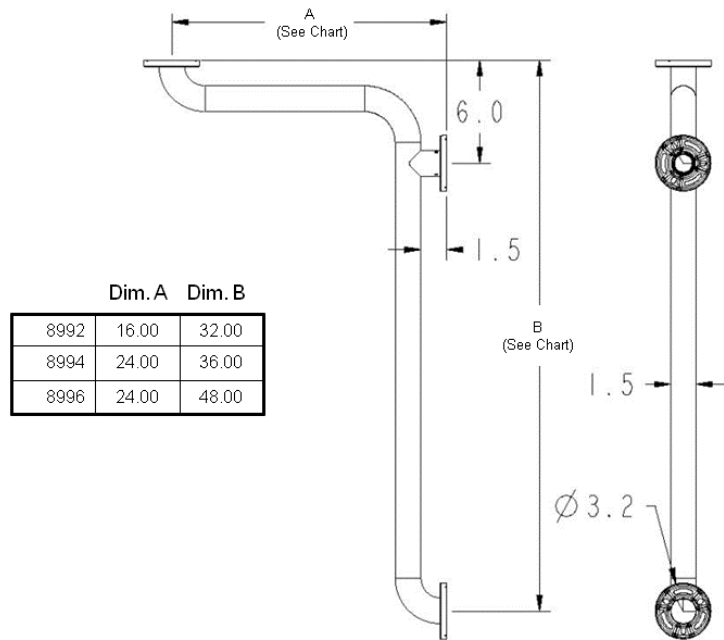

Materials:

Peened Grab Bar rods are formed from 18-gauge type 304 stainless steel tubing with a satin finish

Compliance:

- ADA Compliant
- Tested to 500 lbs. pull

Installation Instructions:

1. Locate wood for backing using stud finder.
2. Locate and level grab bar. Drill 3 x 1/8" diameter pilot holes for both end mounting plates.
3. Mark screw locations on center flange and remove bar from wall.
4. Using hex wrench, loosen 3 set screws and remove center mounting flange.
5. Place center mounting flange on wall and drill 1/8" pilot holes.
6. Install screws.
7. Place L-Bar over center post.
8. Install screws into pre-drilled holes on both ends.
9. Tighten all 3 set screws.
10. Snap on covers.

Stock Number: _____ L-Shape Concealed
Screw Grab Bars Peened Stainless Steel
By Creative Specialties

25300 AL MOEN DRIVE • NORTH OLMSTED, OH 44070
CORPORATE HEADQUARTERS: (800) 321-8809 • FAX: (800) 848-6636
INTERNATIONAL: (440) 962-2000 • FAX: (440) 962-2726
CUSTOMER SERVICE (USA): 800-882-0116 • FAX: (888) 379-2720
WWW.MOEN.COM

©Copyright Creative Specialties International

